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Fitting and wearing instructions	la belle	la belle	junior	headset
<p>Fitting and wearing instructions</p> <p>Insert the la belle listening module (1) on your spectacle arm. Use your glasses without hearing aid - using the push-on spectacle end piece (2). The children's junior hearing system (3) is connected to the headbands using two straps (4). (5) The headset hearing system can be easily adjusted to your head shape via the pull-out adjustment (6).</p>				
				
<p>Setting options on the control unit 1</p> <p>There are 5 control unit versions, each of which differs slightly in its operability.</p> <p>Air conduction</p> <ul style="list-style-type: none"> la belle (with/without external microphone) <p>You can set the following functions yourself:</p> <ul style="list-style-type: none"> Adjust the volume Select the listening programs Switch the unit on/off; see the instructions on the other side 	<p>left</p> 	<p>Without external speaker</p> 	<p>right</p> 	<p>With external speaker on the left</p> 
		<p>Volume control</p> 		<p>With external speaker on the right</p> 
<p>Setting options on the control unit 2</p> <p>Air conduction</p> <ul style="list-style-type: none"> la belle (with/without external microphone) <p>You can set the following functions yourself:</p> <ul style="list-style-type: none"> Select the listening programs Optional: volume control with remote control Switch the unit on/off; see the instructions on the other side 	<p>left</p> 	<p>With external speaker</p> 	<p>right</p> 	<p>Without external speaker</p> 
		<p>Program button</p> 		<p>Volume adjustment is automatic. Manual adjustment possible with optional remote control.</p>
<p>Setting options on the control unit 3</p> <p>Bone conduction</p> <ul style="list-style-type: none"> la belle (control unit above/below) <p>You can set the following functions yourself:</p> <ul style="list-style-type: none"> Adjust the volume Switch the unit on/off; see the instructions on the other side 	<p>left</p> 		<p>right</p> 	<p>No manual adjustment of the hearing programs necessary. The device is already pre-programmed and automatically adapts to the external conditions.</p>
		<p>Volume control</p> 		
<p>Setting options on the control unit 4</p> <p>Bone conduction</p> <ul style="list-style-type: none"> la belle (control unit above/below) junior headset <p>You can set the following functions yourself:</p> <ul style="list-style-type: none"> Adjust the volume Select the listening programs Switch the unit on/off; see the instructions on the other side 	<p>left</p> 		<p>right</p> 	<p>Program button</p> 
		<p>Volume control</p> 		<p>The program button is located on the programming chamber and not on the control unit.</p>
<p>Setting options on the control unit 5</p> <p>Bone conduction</p> <ul style="list-style-type: none"> junior <p>You can set the following functions yourself:</p> <ul style="list-style-type: none"> Select the listening programs Adjust the volume Switch the unit on/off; see the instructions on the other side 	<p>left</p> 		<p>rechts</p> 	<p>Function buttons:</p> <ul style="list-style-type: none"> 1 x start forward one step louder 1 x start back one step quieter 1 x long forward one program further 1 x long back one program back
		<p>Button for volume and program selection</p> 		
		<p>Childlock</p> 		

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